## LIT-Series

LIT2-A3

LIT2-C1

LIT2-C3

LIT2-K2

CE

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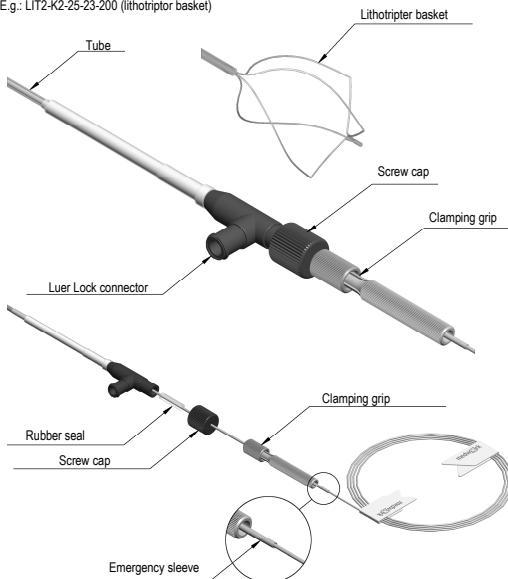
## Instructions for use

### Intended use

The LIT series instruments are used for endoscopic removal of stones from the bile duct and pancreatic duct by mechanical lithotripsy.


### Product features

E.g.: LIT2-K2-25-23-200 (lithotriptor basket)

LIT1-G4-26-185 (transendoscopic mechanical lithotriptor spirals)

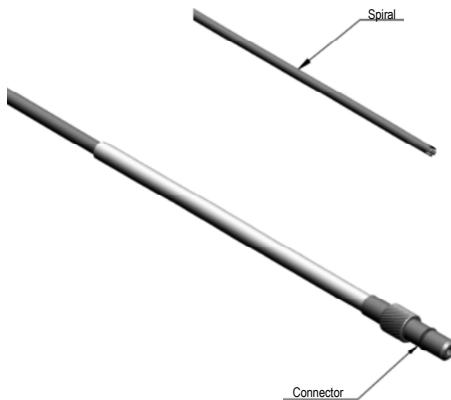LIT1-N4-38-090 (non-endoscopic mechanical lithotriptor/emergency spiral)

### Qualification of user

Use of these instruments requires comprehensive knowledge of the technical principles, clinical applications and risks of gastrointestinal endoscopy. The instruments should only be used by or under the supervision of physicians who are adequately trained and experienced in endoscopic techniques.

### General instructions

Use this instrument only and exclusively for the purposes described in these instructions.

All instruments labelled as unsterile must be prepared in accordance with the appli-

cable medwork preparation instructions before use for the first time.

All medwork instruments should be stored in a dry place that is protected from light. Keep all instructions for use in a safe and easily accessible place.

### Indications

Choledocholithiasis.

### Contraindications

The contraindications for the LIT series are the same as the specific contraindications for endoscopic retrograde cholangiopancreatography.

### Possible complications

Perforation, cholangitis, haemorrhage, aspiration, infection, sepsis, allergic reaction, hypertension, hypotension, respiratory depression or arrest, cardiac dysrhythmia or arrest.

In order to ensure that the examination proceeds smoothly, the diameter of the endoscope working channel and the diameter of the instruments must be matched to one another.

After removal from the packaging, check the instruments for correct function, kinks, fractures, rough surfaces, sharp edges and protrusions. If you discover any damage or malfunction on the instruments, do NOT use them and inform your responsible field service contact or our branch office.

Any person who markets or uses medical devices is required to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Attention!** The lithotriptor baskets, spirals and handles of the LIT series can only be used with LIT and BAS series system components. All BAS series stone extractors are suitable for mechanical lithotripsy, with the exception of the stone extractors marked with this icon.

All BAS series stone extractors are suitable for mechanical lithotripsy, with the exception of the stone extractors marked with this icon. All BAS and LIT series baskets suitable for lithotripsy have a defined, predetermined breaking point.

Remove a stone extraction basket broken off at the shear point very carefully to avoid injuring the mucous membrane. Remove a stone extraction basket broken off at the shear point very carefully to avoid injuring the mucous membrane.

### Instructions for use

Insert the retracted lithotriptor basket into the working channel of the endoscope and gradually slide the catheter forward until the tip is visible in the endoscopic image. Probe the required bile system until behind the stone and carefully capture it under radiological guidance. Carefully pull the collected stone toward the papilla. Strong resistance can point to a stone that is too large or impacted. There is a risk of injuring the ducts. Small stones can be extracted by pulling back the catheter via the papilla. Should the stone not pass the papilla, attempt to crush it manually with the basket. If unsuccessful, the basket must be prepared for a mechanical lithotripsy. Slowly pull the instrument out of the endoscope to prevent contamination of the patient and user with blood and other body fluids, and an associated risk of infection.

### Transendoscopic mechanical lithotripsy

First, flush the instrument with physiological saline solution through the lateral Luer lock connector. Remove the clamping grip and pull the tube off the lightly stretched traction cable. Use the lead line of the lithotriptor basket to slide the flexible lithotriptor spiral (LIT1-G4-26-185) forward transendoscopically to the stone. Insert the traction cable into the lithotriptor handle (LIT2-D-HAN) and attach the emergency spring to the lithotriptor handle with a light rotary movement (see also the LIT2-D-HAN instructions for use). Fix the lead line centred in the chuck. Make sure that the emergency sleeve is outside the chuck (proximal). Now, slowly increase the pressure on the stone by turning the tension screw. If the lithotriptor spiral is under tension, we recommend that you wait a few seconds after each complete rotation of the tension screw before continuing. This allows the force to build up continuously to control how the stone is crushed.

## Non-endoscopic mechanical lithotripsy under radiological control - when using BAS and LIT series stone extraction baskets that can be used for lithotripsy:

First rinse the instrument with saline solution using the Luer-Lock connector on the side. Use a wire cutter to detach handle of the stone extraction basket at the specified cut mark. Then remove the endoscope and the tube. Use the lead line of the stone extraction basket to slide the flexible emergency spiral (LIT1-N4-38-090) forward to the stone. Make sure that the basket is continuously under tension if possible. Insert the traction cable into the lithotriptor handle (LIT2-D-HAN) and attach the emergency spring to the lithotriptor handle with a light rotary movement (see also the LIT2-D-HAN instructions for use). Fix the lead line centred in the chuck. Make sure that the emergency sleeve is outside the chuck (proximal). Now, slowly increase the pressure on the stone by turning the tension screw. If the emergency spiral is under tension, we recommend that you wait a few seconds after each complete rotation of the tension screw before continuing. This allows the force to build up continuously to control how the stone is crushed.

## After completion of the surgical procedure

Reusable instruments must be correctly prepared. Follow the enclosed instructions for reprocessing.

If the instruments can not be used anymore, they must be disposed of in accordance with the relevant applicable hospital and administrative guidelines and in accordance with the relevant applicable legal regulations.

## Disassembly and assembly

To disassemble the instrument, unscrew the clamping grip and remove it from the instrument. Then unscrew the seal. Now slide the exposed pull rod into the tube. The stone extraction basket with the pulling cable can now be withdrawn from the distal end of the tube. Take to prevent contamination during disassembly. After reprocessing, guide the pulling cable into the distal end of the tube until the pull rod emerges from the proximal end. Fix the clamping grip onto the proximal end of the instrument, making contact with the seal. During this step the stone extraction basket is extended. Then close the seal again.

## List of all icons used on medwork products

	Date of manufacture		Use by
	Follow the instructions for use		Caution near pacemakers
	Applied part Type BF		Do not use if packaging is damaged
	Batch code		Not suitable for lithotripsy
	Item number		Contains natural rubber latex
	Packaging unit		Gastroscopy
	Do not reuse		Colonoscopy
	Sterilised using ethylene oxide		Enteroscopy
	Non-sterile		ERCP
	Manufacturer		Medical device
	Sterile barrier system		

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