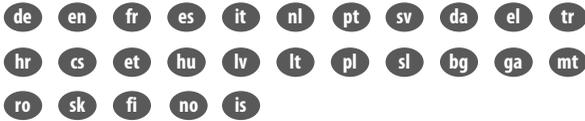
## LIT-Series

LIT1-C3  
LIT1-G4  
LIT1-K2  
LIT1-N4

404450305898

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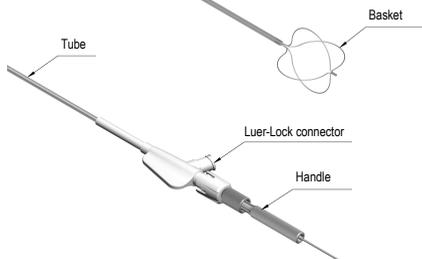
## Intended use

The LIT series instruments are used for endoscopic removal of stones from the bile duct and pancreatic duct by mechanical lithotripsy.


## Product features

E.g.: LIT1-K2-25-23-200 (lithotripter basket)

LIT1-G4-26-185 (transendoscopic mechanical lithotripter spiral)

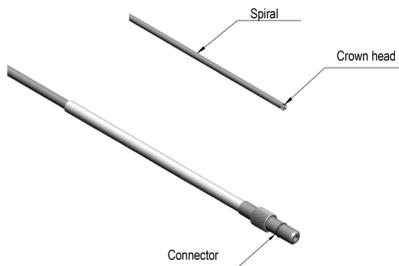LIT1-N4-38-090 (non-endoscopic mechanical lithotripter/emergency spiral)

## Qualification of user

Use of these instruments requires comprehensive knowledge of the technical principles, clinical applications and risks of gastrointestinal endoscopy. The instruments should only be used by or under the supervision of physicians who are adequately trained and experienced in endoscopic techniques.

## General instructions

Use this instrument only and exclusively for the purposes described in these instructions.

Ⓜ **ATTENTION!** Instruments marked with this symbol are intended for single use only and are sterilised with ethylene oxide.

Sterile instruments can be used immediately. Before use, check the "Use by" date on the packaging, as sterile instruments are only allowed to be used up to this date.

Do NOT use the instrument if the sterile packaging has tears or perforations, sealing is not ensured or moisture has penetrated. All FUJIFILM medwork instruments should be stored in a dry place that is protected from light. Keep all instructions for use in a safe and easily accessible place.

FUJIFILM medwork instruments that are labelled as intended for single use only must not be reprocessed, resterilised or reused. Reuse, reprocessing or resterilisation can alter product properties, resulting in functional failure that can endanger the patient's health or lead to disease, injury or death. Reuse, reprocessing or resterilisation furthermore entails the risk of contamination of the patient or the instrument, as well as the risk of cross contamination, including the transmission of infectious diseases. Contamination of the instrument can result in illness, injury or death of the patient.

## Indications

Cholelithiasis.

## Contraindications

The contraindications for the LIT series are the same as the specific contraindications for endoscopic retrograde cholangiopancreatography.

## Possible complications

Perforation, cholangitis, haemorrhage, aspiration, infection, sepsis, allergic reaction, hypertension, hypotension, respiratory depression or arrest, cardiac dysrhythmia or arrest.

## Precautions

In order to ensure that the examination proceeds smoothly, the diameter of the endoscope working channel and the diameter of the instruments must be matched to one another.

After removal from the packaging, check the instruments for correct function, kinks, fractures, rough surfaces, sharp edges and protrusions. If you discover any damage or malfunction on the instruments, do NOT use them and inform your responsible field service contact or our branch office.

Any person who markets or uses medical devices is required to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**Attention!** The lithotripter baskets, spirals and handles of the LIT series can only be used with LIT and BAS series system components. All BAS series stone extractors are suitable for mechanical lithotripsy, with the exception of the stone extractors marked with this  icon. All BAS series stone extractors are suitable for mechanical lithotripsy, with the exception of the stone extractors marked with this icon. All BAS and LIT series baskets suitable for lithotripsy have a defined, predetermined breaking point.

Remove a stone extraction basket broken off at the shear point very carefully to avoid injuring the mucous membrane. Remove a stone extraction basket broken off at the shear point very carefully to avoid injuring the mucous membrane.

## Instructions for use

Insert the retracted lithotripter basket into the working channel of the endoscope and gradually slide the tube forward until the tip is visible in the endoscopic image. Probe the required bile system until behind the stone and carefully capture it under radiological guidance. Carefully pull the collected stone toward the papilla. Strong resistance can point to a stone that is too large or impacted. There is a risk of injuring the ducts. Small stones can be extracted by pulling back the tube via the papilla. Should the stone not pass the papilla, attempt to crush it manually with the basket. If unsuccessful, the basket must be prepared for a mechanical lithotripsy. Pull the instrument out of the endoscope slowly to avoid contaminating the patient and handler with blood and other bodily fluids and subsequent risk of infection.

## Transendoscopic mechanical lithotripsy

First, flush the instrument with physiological saline solution through the lateral Luer lock connector. Now remove the handle and pull the tube off the lightly stretched lead line. Use the lead line of the lithotripter basket to slide the flexible lithotripter spiral (LIT1-G4-26-185) forward transendoscopically to the stone. In this regard, please also observe the instructions for use provided for the LIT series. Make sure that the basket is continuously under tension if possible. Insert the lead line into the lithotripter handle and connect the lithotripter spiral to the handle with a slight rotary motion. Fix the lead line centred in the chuck. Make sure that the fixation bushing is outside the chuck (proximal). Use the X-ray image to ensure that the distal end of the spiral is placed on one axis with the basket before you start the lithotripsy. Now, slowly increase the pressure on the stone by turning the tension screw. If the lithotripter spiral is under tension, we recommend that you wait a few seconds after each complete rotation of the tension screw before continuing. This allows the force to build up continuously to control how the stone is crushed.

## Non-endoscopic mechanical lithotripsy under radiological guidance - when using BAS and LIT series stone extraction baskets that can be used for lithotripsy:

First rinse the instrument with saline solution using the Luer-Lock connector on the side. Now remove the clamping handle, the tube, and the endoscope via the slightly stretched pull cord. Use the lead line of the stone extraction basket to slide the flexible emergency spiral (LIT1-N4-38-090) forward to the stone. In this regard, please also observe the instructions for use provided for the LIT series. Make sure that the basket is continuously under tension if possible. Insert the lead line into the lithotripter handle and connect the emergency spiral to the handle with a slight rotary motion. Fix the lead line centred in the chuck. Make sure that the emergency sleeve is outside the chuck (proximal). Use the X-ray image to ensure that the distal end of the spiral is placed on one axis with the basket before you start the lithotripsy. Now, slowly increase the pressure on the stone by turning the tension screw. If the emergency spiral is under tension, we recommend that you wait a few seconds after each complete rotation of the tension screw before continuing. This allows the force to build up continuously to control how the stone is crushed.

## After completion of the surgical procedure

Instruments intended for single use, including packaging, must be disposed of in accordance with the relevant applicable hospital and administrative guidelines and in accordance with the relevant applicable legal regulations.

### List of all icons used on FUJIFILM medwork products

	Date of manufacture		Use by
	Follow the instructions for use		Caution near pacemakers
	Applied part Type BF		Do not use if packaging is damaged
	Batch code		Not suitable for lithotripsy
	Item number		Contains natural rubber latex
	Packaging unit		Gastroscopy
	Do not reuse		Colonoscopy
	Sterilised using ethylene oxide		Enteroscopy
	Non-sterile		ERCP
	Manufacturer		Medical device
	Sterile barrier system		

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