
FUJIFILM

medwork

LIT-Series

LIT2-D-HAN

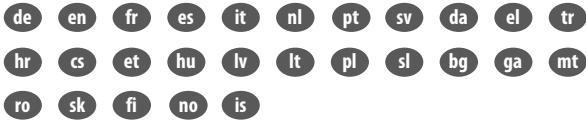CE

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Instructions for use

Intended use

The LIT series instruments are used for endoscopic removal of stones from the bile duct and pancreatic duct by mechanical lithotripsy.

Product features

Figure 1: LIT2-D-HAN


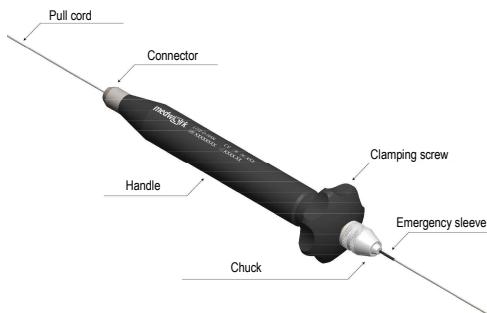
Figure 2: E.g. LIT1-N4-38-090

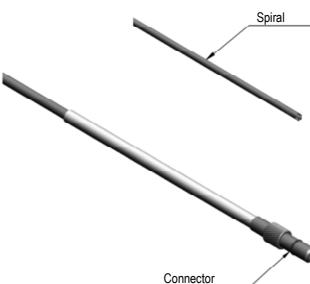Qualification of user

Use of these instruments requires comprehensive knowledge of the technical principles, clinical applications and risks of gastrointestinal endoscopy. The instruments should only be used by or under the supervision of physicians who are adequately trained and experienced in endoscopic techniques.

General instructions

Use this instrument only and exclusively for the purposes described in these instructions.

All instruments labelled as unsterile must be prepared in accordance with the applicable FUJIFILM medwork preparation instructions before use for the first time.

All FUJIFILM medwork instruments should be stored in a dry place that is protected from light. Keep all instructions for use in a safe and easily accessible place.

Indications

Choledocholithiasis.

Contraindications

The contraindications for the LIT series are the same as the specific contraindications for endoscopic retrograde cholangiopancreatography.

Possible complications

Perforation, cholangitis, haemorrhage, aspiration, infection, sepsis, allergic reaction, hypertension, hypotension, respiratory depression or arrest, cardiac arrhythmia or arrest.

Precautions

In order to ensure that the examination proceeds smoothly, the diameter of the endoscope working channel and the diameter of the instruments must be matched to one another.

After removal from the packaging, check the instruments for correct function, kinks,

fractures, rough surfaces, sharp edges and protrusions. If you discover any damage or malfunction on the instruments, do NOT use them and inform your responsible field service contact or our branch office.

Any person who markets or uses medical devices is required to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Attention! The lithotriptor baskets, spirals and handles of the LIT series can only be used with LIT and BAS series system components.

Instructions for use

Open the chuck of the product with the traction cable after placement of the emergency spiral. Then insert the traction cable of the stone extraction basket in the handle and attach the emergency spiral to the handle with a light rotary movement. Carefully note the correct direction of insertion (chuck proximal). Finally, fix the traction cable centred in the chuck by turning the chuck. Make sure that the emergency sleeve or pulling rod is outside the chuck (see BAS and LIT series instructions for use). Use the X-ray image to ensure that the distal end of the spiral is placed on one axis with the basket before you start the lithotripsy. Now slowly increase the pressure on the stone by lightly turning the clamping screw.

After crushing the stone slowly pull the instrument out of the endoscope to prevent contamination of the patient and user with blood and other body fluids, and an associated risk of infection.

After completion of the surgical procedure

Reusable instruments must be correctly prepared. Follow the enclosed instructions for processing.

If the instruments can not be used anymore, they must be disposed of in accordance with the relevant applicable hospital and administrative guidelines and in accordance with the relevant applicable legal regulations.

Disassembly and assembly

After crushing the stone remove the stone extraction basket from the handle by unscrewing the chuck and pull the traction cable out of the handle. For the best possible cleaning results unscrew the chuck and the clamping screw completely from the thread of the handle and send all components for conditioning.

Take to prevent contamination during disassembly.

After cleaning and disinfection mount the handle before starting sterilisation. First screw the clamping screw and then the chuck on the handle.

List of all icons used on FUJIFILM medwork products

	Date of manufacture		Use by
	Follow the instructions for use		Caution near pacemakers
	Applied part Type BF		Do not use if packaging is damaged
	Batch code		Not suitable for lithotripsy
	Item number		Contains natural rubber latex
	Packaging unit		Gastroscopy
	Do not reuse		Colonoscopy
	Sterilised using ethylene oxide		Enteroscopy
	Non-sterile		ERCP
	Manufacturer		Medical device
	Sterile barrier system		

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