FUJIFILM

medwork

LIT-Series _ rotaCRUSH

LIT3-K2

404450305729

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
Instructions for use

Intended use

The LIT series instruments are used for endoscopic removal of stones from the bile duct and pancreatic duct by mechanical lithotripsy.

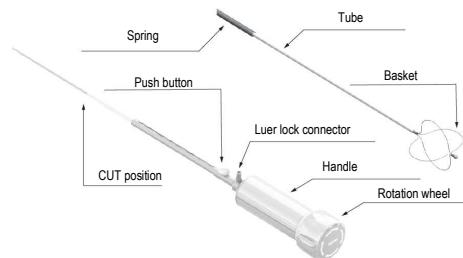Product features

Qualification of user

Use of these instruments requires comprehensive knowledge of the technical principles, clinical applications and risks of gastrointestinal endoscopy. The instruments should only be used by or under the supervision of physicians who are adequately trained and experienced in endoscopic techniques.

General instructions

Use this instrument only and exclusively for the purposes described in these instructions.

ATTENTION! Instruments marked with this symbol are intended for single use only and are sterilised with ethylene oxide.

Sterile instruments can be used immediately. Before use, check the "Use by" date on the packaging, as sterile instruments are only allowed to be used up to this date.

Do NOT use the instrument if the sterile packaging has tears or perforations, sealing is not ensured or moisture has penetrated. All FUJIFILM medwork instruments should be stored in a dry place that is protected from light. Keep all instructions for use in a safe and easily accessible place.

FUJIFILM medwork instruments that are labelled as intended for single use only must not be reprocessed, resterilised or reused. Reuse, reprocessing or resterilisation can alter product properties, resulting in functional failure that can endanger the patient's health or lead to disease, injury or death. Reuse, reprocessing or resterilisation furthermore entails the risk of contamination of the patient or the instrument, as well as the risk of cross contamination, including the transmission of infectious diseases. Contamination of the instrument can result in illness, injury or death of the patient.

Indications

Choledocholithiasis.

Contraindications

The contraindications for the LIT series instruments are the same as the specific contraindications for endoscopic retrograde cholangiopancreatography.

Possible complications

Pancreatitis, cholangitis, sepsis, perforation, haemorrhage, aspiration, fever, infection, hypertension, allergic reaction, respiratory depression or arrest, cardiac arrhythmia or arrest, impaction of the object or objects, aspiration of a foreign body, local inflammations, pressure necroses.

Precautions

In order to ensure that the examination proceeds smoothly, the diameter of the endoscope working channel and the diameter of the instruments must be matched to one another.

After removal from the packaging, check the instruments for correct function, kinks, fractures, rough surfaces, sharp edges and protrusions. If you discover any damage or malfunction on the instruments, DO NOT use them and inform your responsible field service contact or our branch office.

Any person who markets or uses medical devices is required to report any serious incident that has occurred in relation to the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

ATTENTION! Basket wires may fragment and/or stone impaction may occur during a lithotripsy procedure, requiring surgical intervention. If the device malfunctions, while

the stone is in the basket, the device is compatible with the FUJIFILM medwork emergency lithotripsy handle and metal spring. The lithotripter baskets, springs and handles of the LIT series can only be used with LIT series system components. When emergency lithotripsy is needed, cut the tube just below the plastic kink-protection (CUT position).

System preparation

Before use, please make sure that the packaging is not damaged. Lift the sterile bag out of the package, supporting the corner of the handle, and place on a flat surface. Open the sterile bag and pull it away. Undo the carton clips and gently take out the device, starting with the tube/spring and finally the handle. Check the device is in perfect order, no kinks or defects are present. Rotate the black rotating wheel counterclockwise and check the basket fully extends, and rotate back clockwise to initial position. Ensure the spring can be moved forward and backward freely by pressing down the push button first. Flush the device with water through the blue LL connection. Make sure the spring never kinks during use and preparation.

Instructions for Use

Introduce the tube into the working channel of the duodenoscope. Advance the tube until it exits the scope for +/- 1 cm. Cannulate the bile duct, using the elevator of the duodenoscope, and try to pass the tube above the target stone. If there are multiple stones present, always start with the most distal stone. If passage is not possible, then start just below the stone. Rotate the black rotation wheel counterclockwise to the desired basket size, for full 35mm rotate until the stop. Gently move the tube back and forth so the basket moves alongside the targeted stone. Use the rotation effect of the twist'n'CATCH basket by rotating the black rotation wheel (max. 180°). When the targeted stone is caught, move the spring forward as described above, so that the distal crown is pushed against the stone. Verify with fluoroscopy.

Note: After having caught the stone, do not rotate the basket too strong as this might damage the tube. Make sure the spring is locked in one of the 6 locking holes. If not properly locked, the spring can move backward during lithotripsy. Before applying power, make sure that the device is in a straight line with the working channel of the duodenoscope. Slowly rotate the black rotation wheel clockwise, building up pressure, and crushing the stone. If so much crushing power is needed that the rotation becomes uncomfortable, the second power setting can be used. Pull the black rotation wheel back until a click is audible. The second power stage is now set. Now again, slowly rotate the black rotation wheel clockwise, building up additional pressure, and crush the stone. Remove stone fragments with basket.

Prepare the device for a second stone (or fragment extraction)

Pull back the spiral to its original position. Bring the tip of the tube in endoscopic view and rotate the black rotation wheel counterclockwise until it stops. (If you have used power setting II, place fingers on both black and silver part.) When power setting II was used: Rotate only the black rotation wheel counterclockwise until it stops and push back into power setting I. Verify on endoscopic image that the basket is fully expanded. Rotate the black rotation wheel clockwise to its original position and the device is ready for cannulation.

Non-endoscopic mechanical lithotripsy under radiological control:

The endoscope is removed after separation (pliers) of the hand grip from the tube in the CUT position. The flexible emergency spring (LT1-N4-38-090) is advanced via the traction cable to the stone and connected to the lithotripter handle. Also observe the instructions for use of the LIT series in this regard. Use the X-ray image to ensure that the distal end of the spiral is placed on one axis with the basket before you start the lithotripsy. Turning the tension screw on the handle slowly increases the pressure on the stone. Wait a few seconds before continuing to turn the screw because the tension is often sufficient to crush the stone. After crushing the stone, the basket is carefully pulled back out of the duct with the emergency spring. The stone concretions can be retrieved out of the duct with a balloon catheter or a small stone extraction basket.

After completion of the surgical procedure

Instruments intended for single use, including packaging, must be disposed of in accordance with the relevant applicable hospital and administrative guidelines and in accordance with the relevant applicable legal regulations.

List of all icons used on FUJIFILM medwork products

	Date of manufacture		Use by
	Follow the instructions for use		Caution near pacemakers
	Applied part Type BF		Do not use if packaging is damaged
	Batch code		Not suitable for lithotripsy
	Item number		Contains natural rubber latex
	Packaging unit		Gastroscopy
	Do not reuse		Colonoscopy
	Sterilised using ethylene oxide		Enteroscopy
	Non-sterile		ERCP
	Manufacturer		Medical device
	Sterile barrier system		

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